



Criteria	Status (Please check off)	Corrective Action/Comments
AED:		
Open the Lid. Wait for the AED status. NOTE: Observe the change of status indicator to RED. After 5 seconds, verify that the status indicator returns to GREEN.		
Verify that electrodes pads are within their expiry date. Are the pads sealed, undamaged and within expiration date. 2 - 3 -year shelf life		
Listen for voice prompts, check that the display shows the proper corresponding text prompts.		
Verify that batteries are within their expiration date, typically 4 -5 years from date of installation		
Close the lid. Observe the change of status indicator to RED. After 5 seconds, verify that the status indicator returns to GREEN		
Verify that supplies are available for use (razor, mask, gloves,).		
Verify that the unit is clean, undamaged and free from excessive wear.		
Verify that AED is in a visible location with proper signage and it is always easily accessible.		
Verify that all trained responders have current CPR AED certification training.		
Rescue Ready Kit:		
1 x Pocket mask with one-way valve		
1 x Set gloves		
1 x Pair of trauma shears		
1 x Razor (for shaving chest)		
1 x Hand towel		